

Instructions for use


Ileostomy for Aurum Flat, Convex and Profile pouches



Close the outlet on your pouch before applying.



If using a cut-to-fit pouch, cut the hole in the baseplate so it fits the size of your stoma, leaving a 1-2mm clearance. Remove the protective backing from the baseplate, exposing the adhesive part.

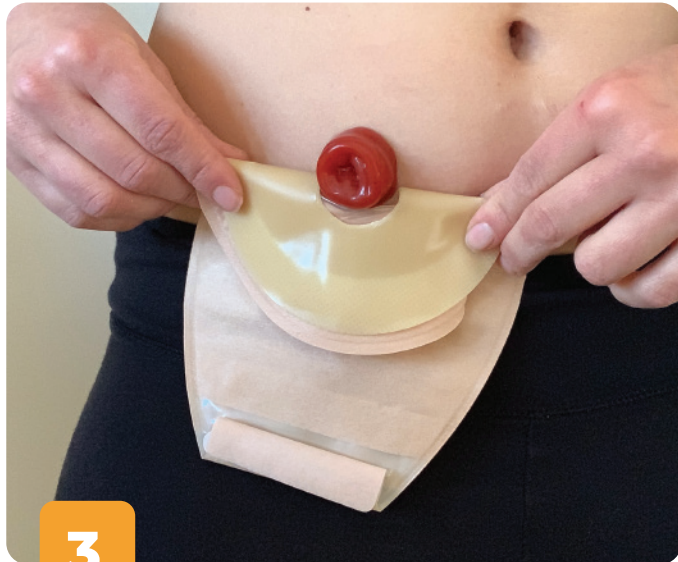

Fold your pouch in half, line up the bottom edge of the hole to the bottom edge of the stoma and press the lower half of pouch into place.



Fold the top over the stoma and press adhesive to the skin. Use your hands to warm the pouch to your skin for about 30 seconds, this will help initial adhesion.

POUCH EMPTYING



5

Peel back Velcro and unfold outlet.



6

Lift easy grip tab and gently squeeze the sides to help empty in the toilet and clean the outlet.



7

Fold up outlet towards you 4 times and secure Velcro.

POUCH REMOVAL



4

Starting at the top of the pouch, gently peel the top edge away from the skin, supporting the skin with the other hand. Peel the pouch downwards, away from the skin.

The use of a medical adhesive remover product is highly recommended.



Aurum Profile
with Manuka Honey



Aurum Profile is most suitable for an ostomate with a hernia, curves or creases.

02-2021



OMNIGON

AU 1800 819 274 info@omnigon.com.au
NZ 0800 440 027 www.omnigon.com.au