

INSTRUCTIONS FOR USE

2-piece drainable pouch


1 PREPARATION



Ensure the area around the stoma is clean and dry. If the pouch is pre-cut to the exact size, please proceed to **step 2**.




If the pouch needs to be cut to size, use the enclosed measuring guide to measure the size and shape of the stoma.

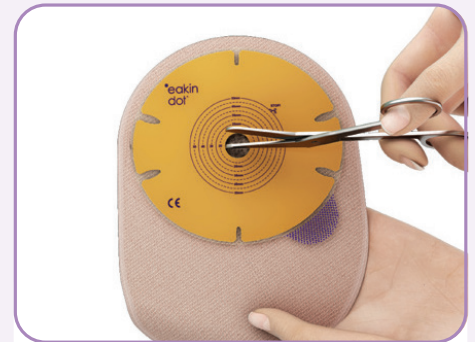


Using the cutting guide on the baseplate, cut a hole to fit the exact pre-measured size and shape of the stoma using curved scissors. Take care not to cut through the pouch film on the other side.

2 APPLICATION

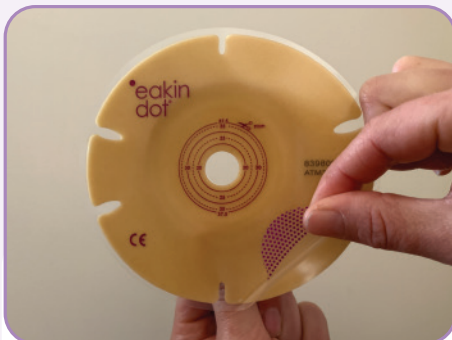

Peel the protective film off the baseplate.

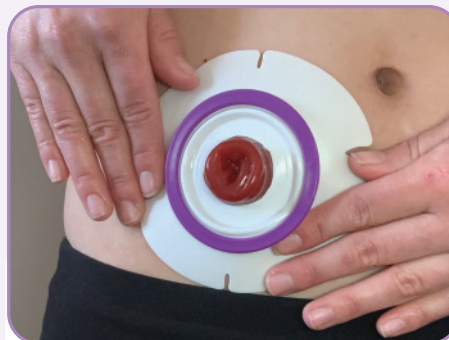

Align the centre hole of the baseplate around the stoma. Starting at the bottom of the baseplate, press gently against your skin, smoothing upwards and outwards to ensure the entire baseplate is securely adhered to the skin.



Use your thumbs to lift the purple ring up away from baseplate, **“pop it up”**.




Connect the pouch to the baseplate.
Squeeze the rings together. You will hear an audible **"click"** as they connect. Walk your fingers around the ring to make sure the pouch is securely attached to the baseplate.



Push the ring back down so it sits flat against the baseplate again.

3 EMPTYING

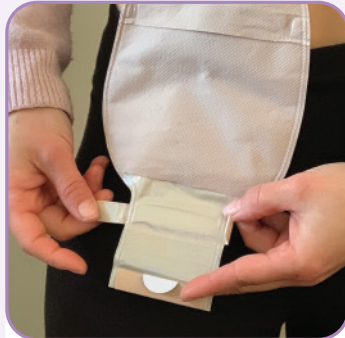

Peel both hook tabs off the Velcro® strip. Unfold the outlet in the position where you want to empty the pouch.



Lift easy grip tap and gently squeeze the sides to help empty and clean the outlet.



To close: Fold the outlet back up towards you and secure by attaching the two hook tabs onto the Velcro® strip.



TIP: For a more discreet look the outlet can be tucked up under the pouch fabric and held in place by the Velcro® dot.

4 REMOVAL




Separate the pouch from the baseplate by holding the baseplate to the skin with one hand. With the other, locate the plastic tab at the top of the pouch. Gently **"peel"** the pouch away from the baseplate in a downward motion.

5 DISPOSAL



Place used products into a waste bag and waste bin.

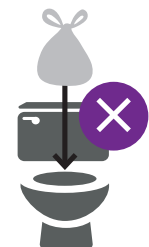

Do not flush the product down the toilet. Dispose of waste products in accordance with local guidelines.



OMNIGON

AU 1800 819 274 info@omnigon.com.au
NZ 0800 440 027 www.omnigon.com.au